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Thank you for choosing our pauto® device

Before using the product, it is important to read this user manual carefully and follow the recommendations and safety instructions.

The instructions in this guide must be followed closely to guarantee safe use of the product. Not following them correctly may result in incorrect measurements, damage to equipment or even injury.

The manufacturer is not responsible for safety, reliability and performance issues, or any anomalies in monitoring, injuries or damage caused by negligence or improper use.

Keep this document safe for future reference. On the manufacturer's website, www.i4life.es, you will find up-to-date product information and contact details as well instruction videos on how to use pauto® and how to get the most out of it to improve your quality of life.

Best wishes and enjoy your pauto®,

the i4life® team

USER'S MANUAL

Manual contents

pauto® description

General information for use

Connecting with carers and family

pauto Application

Care instructions

Disposal

Guarantee

Technical information

This guide is written in line with Regulation (EU) 2017/745 of the European Parliament and Council on medical products, so that you can benefit from the full function of the product.

This product is suitable for domestic, hospital use, health care and rehabilitation. If you are not sure whether it is right for you, you can contact your doctor or the i4life Customer Service Department at the following address info@i4life.es.

pauto® description

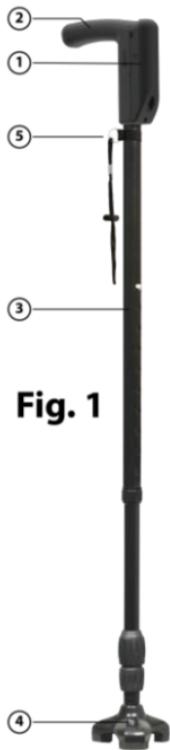
Fig. 1

In the box of the product you have just bought, you will find the pauto® device, shown in figure 1, fully assembled, a pre-activated SIM card and this user manual with information on use and care.

pauto® is shaped like a walking stick, as shown in figure 1. It has a removable box (1) containing the electronics, the communication elements and the battery that powers the system, an ergonomic handle (2), a height-adjustable anodised aluminium cane (3) and a tip (4) or support element. For extra grip, pauto® has a wrist strap (5).

The most important part of pauto is the box (1), pauto®'s brain; figure 2 shows the different parts it is made up of. The box sits within the handle (2) in a slot, attached using a screw (9), which must be loosened to remove the box.

The activation button (6) gives indications on the status of the device using different colours: **Green**, appears when pauto® is transmitting data via the SIM card. **Flashing blue**, when the device is connecting to a mobile via Bluetooth. **Flashing red**, indicates low battery and **constant red** indicates battery charging. pauto® comes with pre-programmed settings that can be


personalised using the pauto® app, as explained in the 'General information for use' section of this manual.

Make sure the box is always charged enough to function correctly. To charge, connect a standard 5V-1000mA micro USB-B charger to the charging port (8), until the red light on the activation button (6) goes off.

To make sure the laser lights (10 and 11) do not shine directly into the eyes, the device has a safety system that deactivates the lights if the device moves away from an upright position.

If you move the stick and it is not vertical, the laser will automatically switch off as a safety measure (figure 3). pauto® can work by connecting to your mobile via Bluetooth. For extra safety, so that pauto® can be located independently, i4life would recommend using a SIM card exclusively for your pauto®.

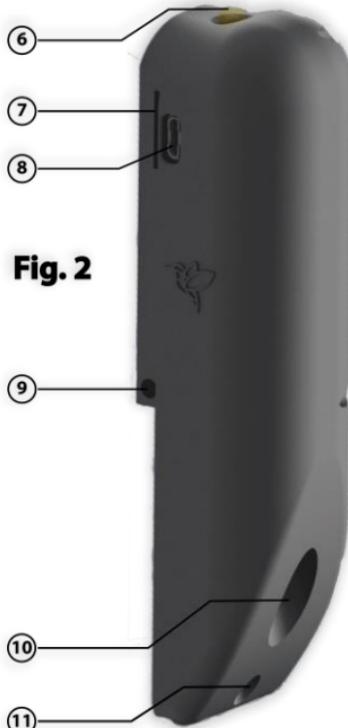To make this easier, we have included a pre-activated card that you can set up easily at the following web address www.i4life.es/pauto.

General information for use

pauto® is designed to locate the user, register their activity and help them to overcome episodes of freezing when walking by means of laser and vibration. There are no known adverse effects from using the pauto®. To get the most from this device, we recommend pairing it with the pauto® app, available on mobiles and pc.

Your pauto® is off for the delivery, plug it into the charger to power on. It is essential to keep an eye on the charge indicator lights to ensure the device does not run out of battery. pauto® is equipped with an activation button (6) that lets you interact and see its status. pauto® comes fully assembled, with all functions switched on for universal use. **We recommend adjusting the settings to suit the user**, through the pauto® app. It also comes minimally charged. **We recommend charging it for two hours before first use.**

Two main modes of the pauto® device are explained below:

- **UNFREEZE MODE.** For those who suffer from freezing of gait when walking, the activation button (6) switches on the lasers (10 and 11) and the vibration, as cues for unfreezing. On this setting, a short


press of the multifunction button (6), pauto® will project a green line and a red dot to help you start walking again. These signals will stay illuminated for 7 seconds and can be activated again by pressing the button when necessary. At the same time, as the lights come on, the handle will start vibrating. This is because it has been proven that the brain responds to haptic stimuli in cases of freezing.

If the user finds themselves disoriented or in danger, pressing the button for a period of 10 seconds will activate an alarm on the mobile phones authorised during the set-up process (carer or family member). If the button is held down for more than 30 seconds, the device will assume this is unintentional and stop the alarm to avoid scares and

false alarms. Remember that if the device (1) deviates more than 22.5° from vertical (figure 3), pauto will automatically stop the lasers to avoid shining directly into the user's eyes, as a safety measure. If

Fig. 3

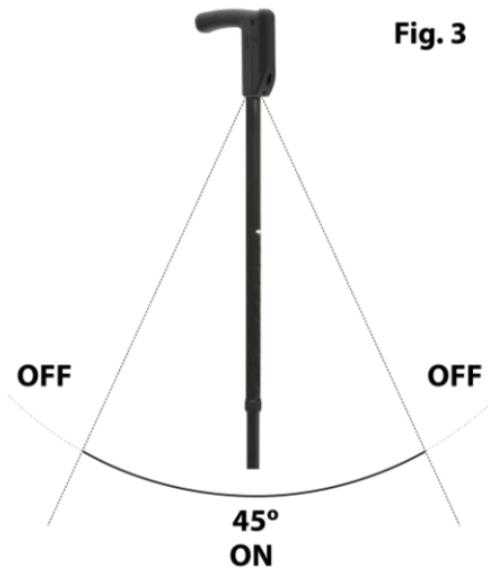the user does not suffer from freezing and only requires the location system, the light signals can be deactivated using the pauto® app.

- **LOCATION MODE.** The user can be located using data if the SIM card is connected or using a smartphone if the SIM has not been activated.

The user can share their location with the permitted phones by pressing the activation button (6) for 10 seconds. pauto® will also automatically send an emergency signal (SMS to phones authorised through the pauto® app) if it detects a fall and the stick does not return to a vertical position in 30 seconds.

To give the user some independence, tracking can be switched off through the pauto® app.

Remember that **the device can only provide these services and peace of mind when it is charged and has signal. Location services and data registering cannot be guaranteed when the pauto has no signal or is not charged.**

Note: the pauto® device has an **airplane mode** that can be switched on and off in the app pauto® or by pushing the button (6) while turning upside-down the stick. As the airplane mode is on, the button (6) will light yellow. To deactivate airplane mode, press and hold the button (6) until it stops showing yellow or use the app.

Height adjustment

This product can be held in either hand. Choose which is best for you and adjust to the correct height. To work out the right height, put on your usual shoes and stand up straight with your elbow slightly bent (20° or 30°). The correct position for the stick will be when the handle is at the same height as your wrist with your arm at rest. Instructions for adjusting are in figure 4: Loosen the fastening ring (14), slide the lower tube (15) to the right position, insert the spring button (13) into the hole, make sure the clip (13) is secure and tighten the fastening ring (14).

Connecting with carers and family

This device allows the user and their carers to track and view the data collected. You must register the carer's phone that you want to be linked to the pauto®(max:4), which you can do using the pauto® app, available for iOS, Android and web.

Fig. 4

Visit www.i4life.es/pauto for more information on the device's connection and autonomy. The device does not locate indoor but it gives the last position. For indoor location an special installation is required.

PAUTO APP

Available on iOS, Android and online at www.i4life/pauto. The software accounts for two different profiles: user profile (the person who will be using the Pauto for walking and everyday life) and the carer profile, who will be linked to pauto® and be able to locate it to find out where the user is in case of emergency. There are photos showing the steps to follow on the web, similar to the mobile app.

USER PROFILE.

You will need to create a user profile online. If you already have an account, log in. On the web app's home page, click on Set Up pauto® on the left side menu.

The Set Up pauto® menu lets you choose the Pauto's functions; Once you have chosen, press the Save Settings to reconfigure your Pauto. If in doubt, begin using your Pauto with all functions and later decide which to keep on and which to turn off.

You will then have to assign a carer (a carer is the person who will be linked to the Pauto through their mobile phone to receive notifications and alerts on its location and the user's activity status).

You can register carers using the About Me section. To comply with Data Protection regulations, once a carer has been added (maximum of 4) and you press the Generate Code button, a 6-digit alphanumeric code will appear; this is the connection code that you will have to give your carers for them to access your data.

If you want to change any of your carers to stop them seeing your data, you can generate a new code and give it to the carers you want to keep.

Finally, as a user, you can always check what version of Pauto you have on your device, update it and check battery power, using the Device Information Menu, which is in the Set Up drop-down. As a user, you can add personal details and medication to your profile.

CARER PROFILE.

After registering on the app, to manage the user who has assigned you as a carer, go to the Carer Code menu on the About User window on the left side bar and enter the code provided by the user.

Once paired with the Pauto device, you can carry out the same actions as the user in the app (medication management) and also

access safe zone management for the user's home (if they leave this area, you will receive a text alert) and using the location menu, you can locate the Pauto device in real time.

To unsubscribe the service, check the website
www.i4life.es/pauto

Care instructions

- Make sure the device is always charged at all times to benefit from all its functions and ensure it is set at the right height to prevent injury.
- Check the main unit and parts regularly to ensure there is no damage or faults. pauto® cannot be used with devices that are not specified in this manual.
- Do not open or repair the device in any way as it may not function correctly thereafter. Not complying with this advice will nullify the guarantee. Any maintenance required must only be carried out by those approved by i4life®.
- Do not put too much weight on the device (maximum 125 kg).
- pauto® assists with walking but it is not designed to support full body weight.

- The box includes a security system that switches off the lights when the angle of the lasers deviates more than 22.5° from a vertical position.
- High frequency portable and mobile communication devices may interfere with pauto®.
- If the device SIM is not activated, user location and data collected from the device will depend on the paired user's mobile phone. This information won't be recorded when the mobile is without signal, switched off, or not near your pauto®.
- When the device operates with the SIM card, location information and data won't be available when the user is in an area with no signal or when the battery runs out.
- If there is a bright light source near to the laser beam (like sunlight or fluorescent tubes), it may be difficult to see the laser clearly.
- pauto® should not be used by people with physical, mental or sensory disabilities, unless they are supervised by someone responsible for their safety or someone who can show them how to operate the device correctly. It is not suitable for children.
- Check regularly that the cane, the clip and the fastening ring are in good condition and that the adjustment holes are still in their original shape.
- Do not look directly into the laser light source when it is illuminated. Exposure to the light source of just 0.25 seconds can cause eye damage.

- Take special care in icy conditions or where there is a risk of slipping. Do not use the pauto® if you think there is a safety risk.
- Do not put the battery or the device on a fire.
- Do not use the pauto® if you are allergic to synthetic products. Do not use with wet or sticky hands, or with a bandage on. Always carry bags or packages in the hand you are not using with pauto®.
- Use a dry cloth to clean the pauto®. Do not use metal brushes or other sharp or pointed cleaning tools. Do not use liquids. Protect your pauto® from liquids, dust, impacts, moisture, extreme temperatures and explosive, corrosive or flammable materials. Do not get it wet or submerge it.
- pauto doesn't locate indoor without an extra installation.

For any questions or if your product is broken, contact info@i4life.es.

Disposal

Disposal of the device should be carried out according to the European Directive on Waste Electrical and Electronic Equipment (WEEE). For more

information, contact the local authority responsible for waste disposal. Used, completely flat batteries must be disposed of in specific collection containers, special waste disposal points or to electronic equipment distributors. Correctly disposing of batteries is a legal requirement.

Guarantee

This product has a 2-year guarantee in line with Royal Decree 1/2007 of 16 November, on manufacturing defects, providing it has not been improperly used.

The guarantee period will begin from the date of purchase. Whenever you require maintenance, you must show your receipt or proof of purchase, so please keep this. During the guarantee period, damage caused by the following will not be repaired for free:

- Incorrect use or handling.
- Not following the instructions for use.
- Modifications or repairs to the product carried out by someone who is not approved by i4life. Natural disasters.
- Defects caused by misuse such as impacts, dropping, overloading, use by more than one person, surges, getting wet, scratched, dirty, etc.

Technical information

- Anodised aluminium rod
- High Density Polymer handle
- Dimensions: 75-100cm x 12cm x 15cm
- Approximate weight: 600 grams (including battery)

- Suitable operating conditions: 10°C-40°C, <75% relative humidity, 700–1060 mmHg atmospheric pressure
- Suitable storage conditions: -40°C to +60°C, <95% relative humidity, 500–1060 mmHg atmospheric pressure
- Power source: Rechargeable lithium ion polymer battery
- Battery: Voltage 3V, power 1100 mAh, useful life: 500 cycles
- Classification IP22, device type B
- Laser classification: Laser type II
- Connectivity: LTE:
B1/B2/B3/B4/B5/B8/B12/B13/B18/B19/B20/B25/B26/B28
- Bluetooth BLE connectivity compatible with beacon mode for indoor location. Outdoors: GPS, GLONASS, Galileo. EGPRS:
850/900/1800/1900MHz

pauto® is a product registered by i4life in the Spanish Office, OEPM. The i4life company reserves the right to update and modify the technical data without prior notice. pauto® complies with the European standard EN60601-1-2 and is subject to special precautions relating to electromagnetic compatibility. Be aware that high frequency portable and mobile communication devices may interfere with this device. You can request more specific information from the i4life Customer Service Department (info@i4life.es).

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